


AMF ALI Panorama Lift

Your vehicle is equipped with an AMF-Bruns ALI Panorama wheelchair access lift. Below is a summary of the main controls of your lift, however we strongly advise that you familiarise yourself with the full operation manual provided.

Lift Operation

Your lift is equipped with an isolator switch which can be used to disconnect the lift from the vehicle battery. It is not essential to do this after using the lift as the current draw is minimal when not in use. Next to the isolator switch is a **resettable circuit breaker**. If a fault should occur, this will render the remote inoperable until it is reset (by pushing the button back in).

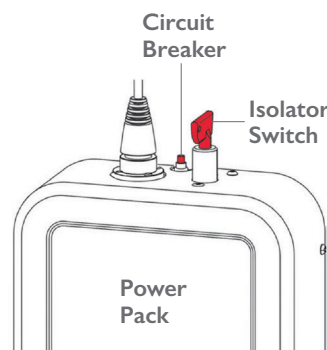


The lift is controlled by a handset on a flying lead. There are four buttons that operate different functions for the lift which are outlined below. If the remote is not illuminated, the **Up** button must be pressed to wake the lift.

- | | | | | | | | |
|--|---|--|---|---|--|--|---|
|  <p>UP</p> | <p>Raises the platform from the floor to the boarding level</p> |  <p>DOWN</p> | <p>Lowers the platform from the boarding level to the floor</p> |  <p>FOLD</p> | <p>Raises the lift to the vertical position for travelling</p> |  <p>UNFOLD</p> | <p>Lowers the lift from the vertical position for use</p> |
|--|---|--|---|---|--|--|---|

Manual Operation

In the event of an electrical failure, the lift can still be operated by manually pumping the hydraulic system.

To raise the lift, insert the manual lever (clipped to the back of the lift next to the power pack) into the pump actuator and pump until the platform reaches the desired position.

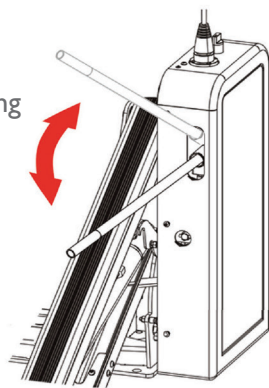

To lower the lift, insert the manual lever over the lowering valve and rotate in an anti-clockwise direction. When the platform has reached the desired position, close the valve by rotating it fully clockwise.

Please note, the valve must be closed before the lift can be raised again.



Important Safety Notes

Do not attempt to drive the vehicle with the lift deployed.

Respect the safe working load which is displayed on the lift.

If you have any questions about your vehicle, please do not hesitate to call us on **01924 442386** or email customerservices@bristolstreetversa.com