


AMF Protektor Restraints

Your vehicle is fitted with a four point wheelchair tie-down and occupant restraint system called Protektor from AMF-Bruns. The Protektor series of restraints are available with a variety of attachments to suit most wheelchairs. The following is an overview of the parts of the Protektor restraint system and how to use them. Please also read and fully understand the manufacturer's instructions provided.


The Protektor system in your vehicle consists of:

- 4x Monofitting floor anchors or airline floor tracking, 4x Retracting restraints (retractors)
- 1x Two-part pelvis belt, 1x Vehicle mounted shoulder belt


Operating the restraints

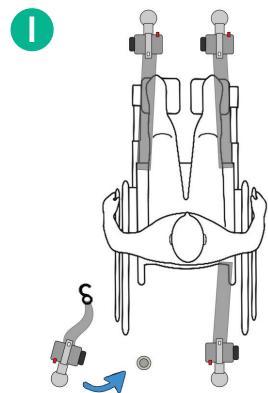


With the wheelchair in position in the vehicle, apply the brakes.
Fit 2x retractors to the front monofittings or in the tracking.
Press the red button to release the retractor.
Extend and attach the restraint to a suitable point on the wheelchair.
Repeat these steps with 2x retractors to the rear.
Apply tension to the rear retractors to secure the wheelchair.

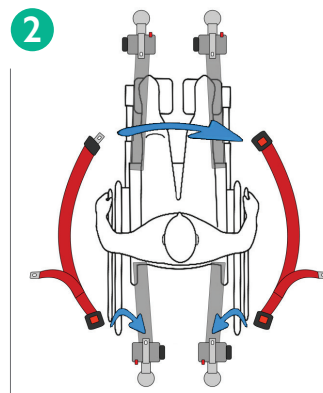

Pass the pelvis belt around the wheelchair occupant.
Attach the buckle on either end of the pelvis belt to the tongue on each rear retractor.
Tighten the pelvis belt as necessary.

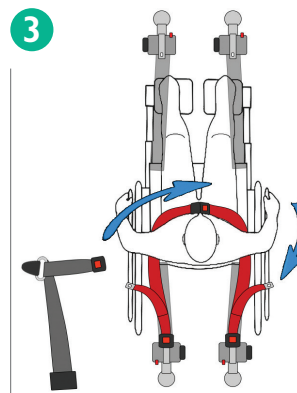


Pass the shoulder belt around the wheelchair occupant.
Attach the buckle to the loose tongue on the pelvis belt.

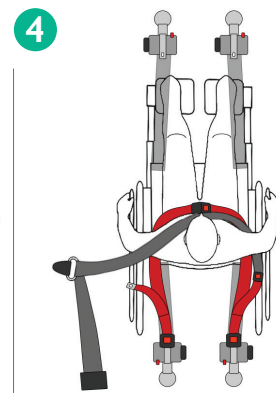

Ensure the pelvis and shoulder belts are comfortable.
The pelvis belt should be pulling down on the hips.
The shoulder belt should sit over the shoulder and not the neck.

Description of the Restraint

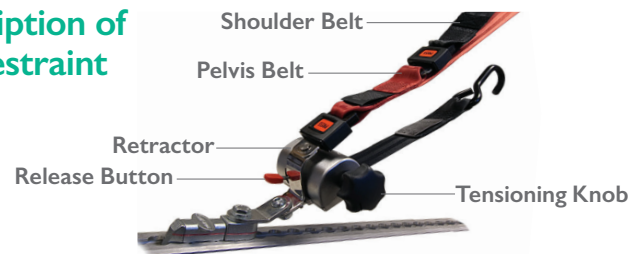

Important Safety Notes

Always take care to ensure the wheelchair is correctly restrained.

Always take care to ensure the wheelchair occupant is correctly restrained.

Please read the Restraint Guidance in your WAV Owner's Handbook for further important information to ensure the safe transportation of the wheelchair and occupant.

If you have any questions about your vehicle, please do not hesitate to call us on **01924 442386** or email customerservices@bristolstreetversa.com